

Adhesives Options

It is important to keep your Eversense Smart Transmitter firmly in place over the sensor. Below are commonly used adhesive supplements.

General Tips

- Start with clean, dry skin. Remove any lotion or soap prior to placing adhesive patch and smart transmitter.
- Your kit comes with 150 white and 30 clear adhesive patches and you should try both. The white is perforated and may be better at wicking away moisture. The clear is optimal for discretion. Both types are gentle and use a medical grade, non-reactive silicone adhesive. It does not include natural rubber latex.
- If any skin irritation persists, consult with your health care provider for additional treatment options.



Other Available Products and Options

Product	How It Works	May provide extra skin adherence	May help decrease skin irritation	Considerations
SKIN-PREP® or No-Sting SKIN PREP (Smith & Nephew) 	<ul style="list-style-type: none"> • Creates thin, tacky layer on the skin • Provides a barrier to skin moisture 	✓	✓	<ul style="list-style-type: none"> • Use only after incision is fully healed • Let dry completely before applying the adhesive patch and smart transmitter
SKIN TAC™ Adhesive Barrier Wipe 	<ul style="list-style-type: none"> • Creates thin, tacky layer on the skin • Provides a barrier to skin moisture 	✓	✓	<ul style="list-style-type: none"> • Use only after incision is fully healed • Let dry completely before applying the adhesive patch and smart transmitter • May need to use an alcohol wipe to assist with removing product
Smith & Nephew IV 3000 1-Hand transparent dressing 	<ul style="list-style-type: none"> • Physical barrier between skin and adhesive patch 	✓	✓	<ul style="list-style-type: none"> • Low allergy adhesive, physical barrier between skin and adhesive patch. • Comes in different sizes: 4" x 5" and 4" x 8"



Product	How It Works	May provide extra skin adherence	May help decrease skin irritation	Considerations
Benadryl® Antihistamine Spray 	<ul style="list-style-type: none"> Creates thin barrier on the skin Calms the skin 	✓	✓	<ul style="list-style-type: none"> Use only after incision is fully healed Apply thin layer, wait 10 minutes, and then apply adhesive patch and smart transmitter
Non-adhesive Wraps (Coban, Ace) 	<ul style="list-style-type: none"> Extra support for holding smart transmitter and adhesive patch in place 	✓		<ul style="list-style-type: none"> Do not wrap too tightly to avoid excessive tissue compression
KT Tape® Kinesiology Therapeutic Tape 	<ul style="list-style-type: none"> Extra support for holding smart transmitter and adhesive patch in place Gentle on skin 	✓		<ul style="list-style-type: none"> Elastic sports tape may be stretched as needed to fit arm comfortably Gentle on skin
GRIFGRIPS®  Strip Grip Wing Grip	<ul style="list-style-type: none"> Extra support for holding smart transmitter and adhesive patch in place Specifically sized for Eversense Smart Transmitter 	✓		<ul style="list-style-type: none"> To view colors, patterns, and designs, go to https://www.grifgrips.com/ Note: Available only in the USA
EversenseFix 	<ul style="list-style-type: none"> Extra support for holding smart transmitter and adhesive patch in place Specifically sized for Eversense Smart Transmitter 	✓		<ul style="list-style-type: none"> Order at https://www.diastuff.de/en/produkt/eversensefix-the-optimal-fixation-for-your-eversense-xl-transmitter/ Note: Available in Europe only
Unscented solid or spray antiperspirant (not deodorant) 	<ul style="list-style-type: none"> Prevents moisture from perspiration 	✓		<ul style="list-style-type: none"> Use only after incision is fully healed Apply thin layer, wait 10 minutes, and then apply adhesive patch and smart transmitter

* **References:** Messer L, Beatson C. Preserving Skin Integrity with Chronic Device Use in Diabetes. *Diabetes Technology & Therapeutics* 2018; 20 Suppl 2. Tsai, A. Tips for Tackling Adhesive Irritation and Allergies. *Diabetes Forecast*, 2017 March.

© 2024 Senseonics, Inc.
All rights reserved.
© 2024 Ascensia Diabetes Care Holdings AG.
All rights reserved.



Distributed by Ascensia Diabetes Care
5 Wood Hollow Road
Parsippany, NJ 07054
844-SENSE4U (736-7348)
ascensiadabetes.com/eversense

Manufactured by Senseonics, Inc.
20451 Seneca Meadows Parkway
Germantown, MD 20876-7005 USA

The Eversense® E3 Continuous Glucose Monitoring (CGM) System is indicated for continually measuring glucose levels for up to 180 days in persons with diabetes age 18 and older. The system is indicated for use to replace fingerstick blood glucose (BG) measurements for diabetes treatment decisions. Fingerstick BG measurements are still required for calibration primarily one time a day after day 21, and when symptoms do not match CGM information or when taking medications of the tetracycline class. The sensor insertion and removal procedures are performed by a health care provider. The Eversense E3 CGM System is a prescription device; patients should talk to their health care provider to learn more.

For safety information, see <https://www.ascensiadabetes.com/eversense/safety-info/>
Eversense, Eversense E3 Continuous Glucose Monitoring and the Eversense logo are trademarks of Senseonics, Incorporated. Ascensia and the Ascensia Diabetes Care logo are trademarks and/or registered trademarks of Ascensia Diabetes Care Holdings AG. Android is a trademark of Google LLC. Apple Watch is a product of Apple, Inc., and may be separately purchased from an authorized Apple retailer. Apple Watch is not included with the Eversense CGM System. All other trademarks are properties of their respective owners and are used solely for informative purposes. No relationship or endorsement should be inferred or implied. PP-SENS-GBL-0097