



GETTING READY

For Your CGM Appointment

Congratulations on beginning your journey with Eversense 365.
Let's get you started!

BEFORE Your Appointment

Follow the *Getting Started Steps* in the *Eversense Quick Reference Guide* (eversensecgm.com/user-guides)

- From your compatible mobile device, download the Eversense 365 App from the App Store® or Google Play™
 - Download this app → 
- Tap the Eversense icon to launch the app, and follow the prompts to create your Eversense account
 - On the Welcome screen, choose "I do not have a Smart Transmitter"
 - On the Unit of Measurement screen, leave it set at mg/dL
 - Write down your account username (email) and password:
email: _____
password: _____
- After creating your account, you will receive an email. Click on the link in email within 72 hours to activate your account.

DAY OF Your Appointment

- Wear a loose, short-sleeved or sleeveless shirt
- What to bring:
 - Eversense account username (email) and password
 - Your mobile device, fully charged and with the Eversense 365 App installed.
- You can return to your normal activities. However, to allow proper healing, it's not recommended that you soak in a tub, swim, or do strenuous activities for five days.

Questions? We're here for you!
Eversense Customer Care:
1-844-736-7348 (1-844-SENSE4U)



Sign up for your Eversense Product Training today!
Our clinical education team will guide you through everything you need to know to confidently prepare for your Eversense sensor placement. Don't miss out!

▶ **Scan here** or visit bit.ly/eversense-virtual-classes to view upcoming dates and secure your spot!

© 2024 Senseonics, Inc.
All rights reserved.
© 2024 Ascensia Diabetes
Care Holdings AG.
All rights reserved.

Distributed by Ascensia
Diabetes Care
5 Wood Hollow Road
Parsippany, NJ 07054
844-SENSE4U (736-7348)
eversensecgm.com

Manufactured by Senseonics, Inc.
20451 Seneca Meadows Parkway
Germantown, MD 20876-7005 USA



The Eversense® 365 Continuous Glucose Monitoring (CGM) System is indicated for continually measuring glucose levels for up to one year in people (18 years and older) with diabetes. The system is indicated for use to replace fingerstick blood glucose (BG) measurements for diabetes treatment decisions. Fingerstick BG measurements are required for calibration one time a week after day 13, and when symptoms do not match CGM information or when taking medications of the tetracycline class. The sensor insertion and removal procedures are performed by a healthcare provider. The Eversense 365 CGM System is a prescription device; patients should talk to their healthcare provider to learn more.

For safety information, see bit.ly/eversensesafety

Eversense, Eversense 365 Continuous Glucose Monitoring, and the Eversense logo are trademarks of Senseonics, Incorporated. Ascensia, the Ascensia Diabetes Care logo are trademarks and/or registered trademarks of Ascensia Diabetes Care Holdings AG. All other trademarks are properties of their respective owners and are used solely for informative purposes. No relationship or endorsement should be inferred or implied.
PP-SENS-GBL-0128